



207 Perry Parkway, Suite 2
Gaithersburg, MD 20877
240-404-9045
www.panacea-labs.com

TK Sensesm Test Order Form Information and Requirements

TK Sensesm must be ordered by a physician. The ordering physician's contact information must be recorded on the test order form.

If shipping materials are unavailable at the site where the sample will be drawn, contact Panacea Laboratories at 240-404-9045.

Specimen Information and Requirements

10 ml of whole blood should be drawn into two 6 ml EDTA (lavender top) tubes *OR* four 3 ml EDTA (lavender top) tubes.

The EDTA tubes must be labeled completely with the patient's identification.

The test order form must be completed in its entirety.

Fasting or any other specific restrictions are unnecessary prior to the blood draw.

Shipping Information and Requirements

Specimens must arrive at Panacea Laboratories the next morning after collection.

Ship Monday thru Thursday **ONLY!** Please note: Specimens will not be accepted on weekends or federal holidays.

Ship specimens via FedEx Priority Overnight.

Specimens are shipped on ice pack(s) following the enclosed shipping instructions.
DO NOT FREEZE / DO NOT PLACE ON DRY ICE

Payment Information and Requirements

The cost of TK Sensesm is \$500.00 USD.

Payment can be made by check (payable to Panacea Pharmaceuticals) or credit card (option available soon). Please be sure to fill out the payment section of the test order form when submitting a specimen.



TK Sensesm Test Order Form Specimen Collection and Shipping Instructions

Specimen Collection Instructions

1. All patient specimens should be considered potentially infectious and should be handled using universal precautions in accordance with your institute's standard procedures.
2. Label EDTA tubes completely with the patient's full name, unique identifier (if available), date of birth, sex, and the date and time that the specimen was drawn.
3. Collect 10 ml of whole blood using two labeled 6 ml EDTA (lavender top) tubes *OR* four labeled 3 ml EDTA (lavender top) tubes.
4. Maintain all EDTA tubes at room temperature or refrigerate until shipped.
5. Complete the test order form. Be sure to include the ordering physician's name and contact information on the test order form to ensure accurate test result communication. Ensure that all information is correct and complete.

Shipping Instructions

1. Ensure that the test order form is completed in its entirety.
2. Place all EDTA tubes in a biohazard specimen bag along with an absorbent pad.
3. Fold and place test order form in the side pocket of the biohazard specimen bag.
4. Roll the specimen's biohazard specimen bag around the specimens and seal the bag (allowing the excess air to escape).
5. Place ice pack(s) at bottom of Styrofoam cooler.
6. Place specimens on top of ice pack(s) and cover Styrofoam cooler.
7. Insert Styrofoam cooler into cardboard shipping box and seal cardboard shipping box with sealing tape.
8. Label cardboard shipping box with UN3373 Category B Biological Substance label.
9. Fasten completed FedEx airbill to top of sealed cardboard shipping box.
10. Specimens should be shipped using FedEx Priority Overnight. Please contact Panacea Laboratories at 240-404-9045 for any specimen transport questions you may have.
11. Contact 800-GO-FEDEX (800-463-3339) to arrange FedEx pick up. For any other arrangements, contact Panacea Laboratories 240-404-9045.



TK Sensesm Test Order Form

Specimen Information

Date Specimen Obtained: _____ (mm/dd/yyyy)
Time Specimen Obtained: _____ (24-hr Military Time)
Tubes shipped:
 Two 6 ml EDTA tubes (lavender top)
OR
 Four 3 ml EDTA containing tubes (lavender top)

Patient Information

Name: Last _____ First _____ MI _____

Unique Identifier (if available): _____

Date of Birth: _____ (mm/dd/yyyy)

Sex: Male Female

Physician Information

Name: Last _____ First _____ MI _____

Telephone Number (with area code): _____

Fax Number (with area code): _____

E-mail Address: _____

Preferred Address for Correspondence:

Facility Name: _____

Street Address: _____

City: _____ State: _____ Zip Code: _____

Payment Information

Payment can be made by check (payable to Panacea Pharmaceuticals) or credit card (option available soon). The cost of TK Sensesm is \$500.00 USD.

- Check- Enclosed Check Made Payable to Panacea Pharmaceuticals
 Credit Card- Option Available Soon

Panacea Laboratories is a division of Panacea Pharmaceuticals, Inc.

Panacea Laboratories developed TK Sensesm and has determined its analytical performance characteristics. Panacea Laboratories is regulated under the Clinical Laboratory Improvements Amendments of 1988 (CLIA) and is qualified to perform high-complexity testing. TK Sensesm is intended to be used for clinical purposes. The FDA has determined that clinical laboratory tests such as TK Sensesm do not require prior approval or clearance. CLIA# 21D1057682 MD State #1352